

prosthetic.line

CM Connect

CLINICAL SCREW  
RECOMMENDED TORQUE.

# CM Connect

## Clinical screw recommended torque.

CONNECTION	PLATFORM SIZE	REF. N°	RECOMMENDED TORQUE [Ncm]	ORIGINAL EQUIVALENT REF. N°
<b>EXT.HEX.UNIVERSAL</b> (Brånemark® System)	NP	41.19.027	35	Nobel Biocare® 31168
	NP	41.19.227	35	Nobel Biocare® 29282
	NP	41.19.327	35	-
	NP	41.19.101	35	-
	RP	41.19.028	35	Nobel Biocare® 29290
	RP	41.19.228	35	Nobel Biocare® 29283
	RP	41.19.328	35	-
	WP	41.19.029	35	Nobel Biocare® 31169
	WP	41.19.229	35	Nobel Biocare® 29284
	WP	41.19.329	35	-
	NP on implant for Multi-unit®	41.19.501	35	Nobel Biocare® 29194
	RP on implant for Multi-unit®	41.19.502	35	Nobel Biocare® 29195
<b>TRI – LOBE</b> (Nobel Replace® Select™)	NP	41.19.004	35	Nobel Biocare® 31170
	NP	41.19.204	35	Nobel Biocare® 36818
	NP	41.19.304	35	-
	RP/WP	41.19.005	35	Nobel Biocare® 29293
	RP/WP	41.19.205	35	Nobel Biocare® 29475
	RP/WP	41.19.305	35	-
	NP on implant for Multi-unit®	41.19.504	35	Nobel Biocare® 29242
	RP on implant for Multi-unit®	41.19.505	35	Nobel Biocare® 29243
<b>ACTIVE HEX</b> (Nobel Active®)	NP	41.19.027	35	Nobel Biocare® 31168
	NP	41.19.227	35	Nobel Biocare® 29282
	NP	41.19.101	35	-
	NP	41.19.327	35	-
	RP	41.19.030	35	Nobel Biocare® 28815
	RP	41.19.230	35	Nobel Biocare® 36918
	RP	41.19.328	35	-
	NP on implant for Multi-unit®	41.19.507	35	Nobel Biocare® 36892
	RP on implant for Multi-unit®	41.19.508	35	Nobel Biocare® 36893
<b>OCTAGON</b> (Straumann® Soft Tissue Level®)	RN/WN on implant	41.19.026	35	STRAUMANN® AG 048.356
	RN/WN on implant	41.19.102	35	STRAUMANN® AG 049.181
	RN/WN on implant	41.19.328	35	-
	RN/WN on synOcta® abutment	41.19.009	15	STRAUMANN® AG 048.350
	RN/WN on synOcta® abutment	41.19.103	15	-
<b>CONICAL BL</b> (Straumann Bone Level®)	NC on implant	41.19.032	35	STRAUMANN® AG 025.2908
	RC on implant	41.19.033	35	STRAUMANN® AG 025.4908
	NC/RC on implant	41.19.037	35	STRAUMANN® AG 025.4900
	NC/RC on implant	41.19.104	35	STRAUMANN® AG 025.4906
	NC/RC on implant	41.19.311	20	-
	NC/RC on implant for Multi-unit®	41.19.544	35	-
	NC/RC on Multi-base® abutment	41.19.041	15	STRAUMANN® AG 025.0900-04
	NC/RC on Multi-base® abutment	41.19.105	15	-
<b>INTERNAL HEX CONIC</b> (ASTRA TECH™)	RP on implant	41.19.016	20	DENTSPLY® 24449
	WP on implant	41.19.017	25	DENTSPLY® 24209
	UniAbutment® 20°/ 45°	41.19.018	15	DENTSPLY® 22435
<b>INTERNAL HEX FD (XIVE®)</b>	NP/RP/WP	41.19.024	24	DENTSPLY® 454305

CONNECTION	PLATFORM SIZE	REF. N°	RECOMMENDED TORQUE [Ncm]	ORIGINAL EQUIVALENT REF. N°
<b>INTERNAL HEX USA</b> (Zimmer Screw Vent™ MIS® Implants)	NP/RP/WP	41.19.012	30	Zimmer® URS2
	NP/RP/WP	41.19.012L	30	-
	NP/RP/WP	41.19.042	30	Zimmer® MHLAS / MIS® MN-S0160
	NP/RP/WP	41.19.312	30	-
	NP (MIS®)	41.19.327	35	-
	NP on implant for Multi-unit®	41.19.519	30	-
	RP on Multi-unit® abutment Ø4.8	41.19.110	15	-
	RP on Multi-unit® abutment Ø4.8	41.19.300	15	-
<b>EXT HEX USA</b> (3i® Osseotite®)	NP/RP/WP	41.19.034	20	BIOMET 3i® UNIHT
	RP on implant for Multi-unit®	41.19.512	20	-
	Low Profile® Abutment Ø4.8	41.19.106	10	BIOMET 3i® LPCTSH
<b>INTERNAL HEX CLICK</b> (3i® Certain®)	NP/RP/WP	41.19.011	20	BIOMET 3i® IUNIHT
	NP/RP/WP	41.19.311	20	-
	Low Profile® Abutment Ø4.8	41.19.106	10	BIOMET 3i® LPCTSH
<b>EXTERNAL HEX BH</b> (BioHorizons®)	NP	41.19.013	30	BioHorizons® 130-050
	NP	41.19.312	30	-
	RP/WP	41.19.014	30	BioHorizons® 140-050
	RP/WP	41.19.328	35	-
<b>MULTI-UNIT®</b>	Multi-unit® RP abutment Ø4.8	41.19.006	15	Nobel Biocare® 29287
	Low Profile® Abutment Ø4.8	41.19.106	10	BIOMET 3i® LPCTSH
	Multi-unit® RP abutment Ø4.8	41.19.206	15	Nobel Biocare® 29285
	Multi-unit® RP abutment Ø4.8	41.19.306	15	-
	Multi-unit® RP abutment Ø4.8	41.19.110	15	-
	MIS® Multi-unit® RP abutment Ø4.8	41.19.300	15	-
	Multi-unit® WP abutment Ø6.0	41.19.031	15	Nobel Biocare® 31163
	Multi-unit® WP abutment Ø6.0	41.19.231	15	Nobel Biocare® 29286
	Multi-unit® WP abutment Ø6.0	41.19.331	15	-
<b>ANKYLOS® C/X</b>	All platforms	41.19.048	15	-
	All platforms	41.19.050	15	-
<b>SPECIAL DESIGN</b> (Dyna® System)	Universal 3.6	41.19.038	32	Dyna Dental 31AS1
	Octa	41.19.039	30	Dyna Dental 81AS0
	Extension Level Octa	41.19.040	30	Dyna Dental 82ES0

All CM Connect products are manufactured according to ISO 9001:2008 and ISO 13485:2012 and bear the CE marking. 3i® and 3i® Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA. ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XiVE® and FRIADENT® and UniAbutment® are registered trademarks of DENTSPLY Implants. BioHorizons® registered trademark of BioHorizons, INC. Brånemark System®, NobelReplace® and Replace Select™ and NobelActive™ and NobelReplace® CC and NobelSpeedy™ and Multi-unit® registered trademarks of Nobel Biocare AB, Sweden

ITI®, Straumann® and synOcta® registered trademarks of Straumann Holding AG, Switzerland. MIS® registered trademark of MIS Implant Technologies Ltd. Zimmer Screw-Vent® and SwissPlus® registered trademarks of Zimmer Dental INC, USA. Dyna® registered trademark of Dyna Dental Engineering BV

Locator® registered trademark of Zest Anchors INC. The type, reference and connection of each element should be checked. Some CM Connect items may not be available in your country. These instructions replace all previous editions. For any detailed information about CM Connect products please contact your local distributor.

